

BATH POSITIONING CHAIR

USER MANUAL





Product colour may vary in this document.

IMPORTANT

These instructions provide important information for the safe use and maintenance of the Bath Positioning Chair. Please take the time to read the below; product must be used in accordance with the instructions to avoid any harm to the user.

Intended use

The Bath Positioning Chair is a bath chair that provides support for children and young adults during the bath or shower. The bath chair is for indoor use in a bathroom environment.

Package Contents

The bath Positioning Chair is completely assembled, you will only need to position the backrest, calf support and bath chair legs appropriately before use. Please check that you have all the accessories you ordered.

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled and disposed of safely.

Maintenance

This product should be inspected, maintained, cleaned and/or washed routinely. Discontinue the use of this bath chair if it becomes damaged, broken or disassembled. Contact your supplier for qualified repair or replacement. Do not alter or repair by unqualified people. In the case that modifications or repairs are made without having received written permission from the vendor, the warranty will be void.




ADJUSTMENTS




WARNING

- Do not let the user play with this product.
- Ensure that all the locking devices are adjusted correctly and tightened before use.
- Adjust the bath chair to a comfortable level before placing the user in it. If the user feels uncomfortable, remove them from the chair as quickly as is safely possible.
- With the exception of the calf support recline adjustment, the bath chair should not be adjusted while it is in use or occupied.


Adjust the backrest

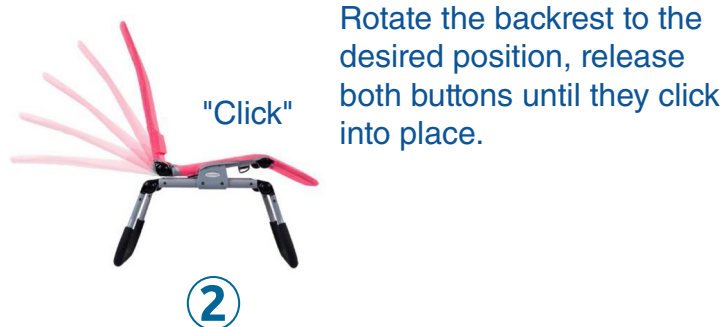


Adjust the calf support

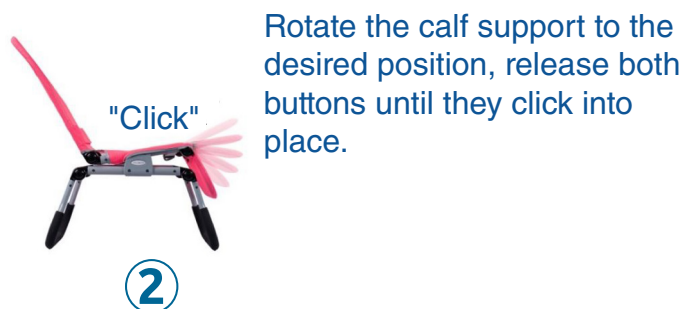

Adjust bath chair back leg

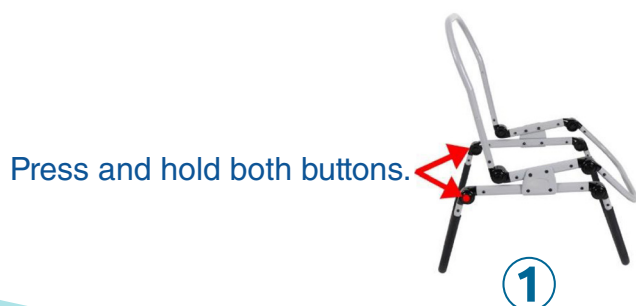


Adjust bath chair back leg and front


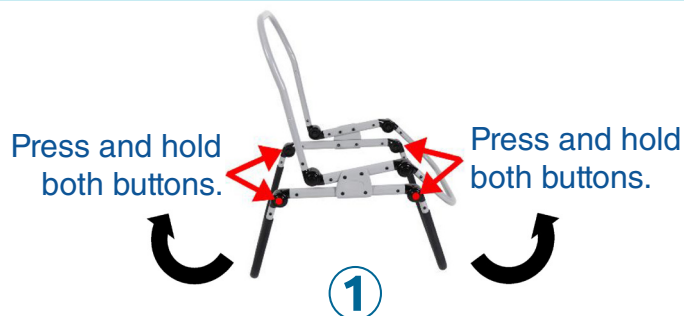


WARNING

The bath chair is likely to tip over if both the front and back legs are positioned at or below 90 degrees.

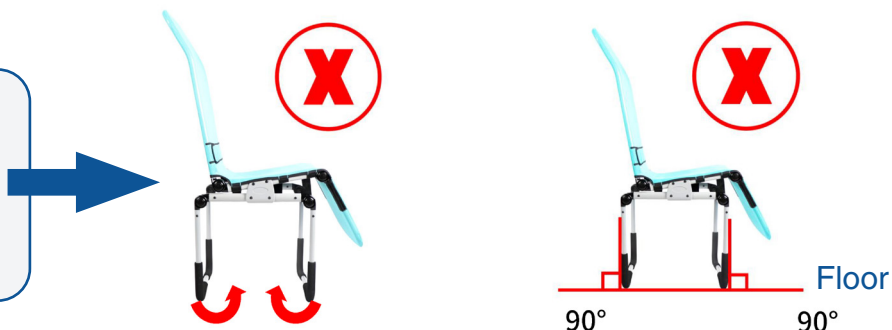

Folding bath chair

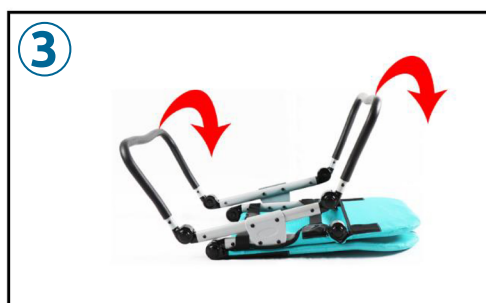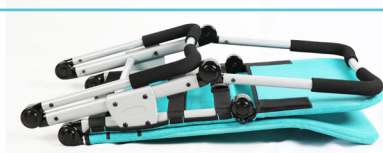

Rotate the bath chair 180°


Removing the bath chair cover



TUB / SHOWER STAND




Assembling the tub / shower stand

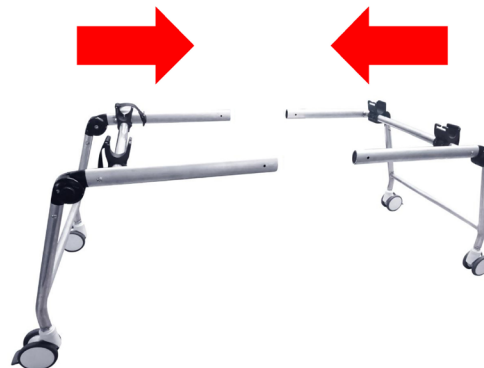



To assemble, press and hold the button on the slender tube, then insert it into the larger tube until the button pops out of the hole in the thick tube. Once you've ensured the stand is securely locked and stable, it is ready for use.

WARNING

If the legs of the tub/shower stand are at or below 90 degrees, there is a high risk of it tipping over.
Maximum weight : 200 lbs.

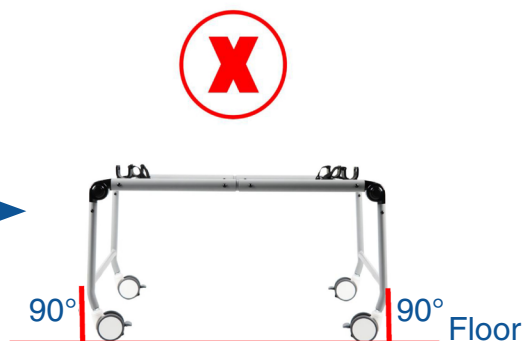

Folding the tub / shower stand

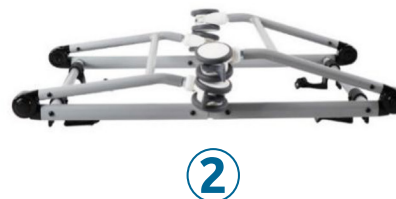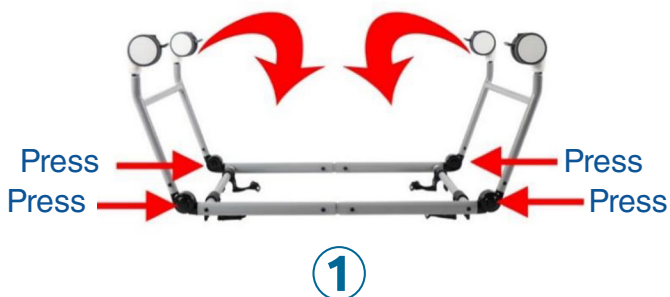

Attaching the bath chair to the tub / shower stand



WARNING

- The tub/shower stand is only to be used in combination with Bath Positioning Chair. The tub/shower stand is only intended for bathing, no other uses are permitted.
- Check that the bath chair is properly attached and stable on the tub/shower stand before usage.
- NEVER lift the tub/shower stand with a user in it.
- Inspect the tub/shower stand for damage and wear regularly, and discontinue use if any part is broken, torn, or missing. Repair or replace any damaged or worn parts as soon as possible.

Shower stand casters

WARNING

To prevent falls or injuries, always lock all the casters when placing or removing the user.



Cleaning

The tub/shower stand can be wiped clean with a moist cloth and a mild cleaning agent.

ACCESSORIES

Head Lateral Support

Chest Belt

Leg Straps

Calf Straps



Fixed Pelvic Belt (M/L size)

WARNING


- Accessories are not intended for prolonged periods of support.
- Ensure that all accessories are adjusted for comfort and security.
- Do not place the chest belt and/or the pelvic belt around a user's neck.

HOW TO ATTACH THE ACCESSORIES TO THE BATH CHAIR

Head lateral straps



①

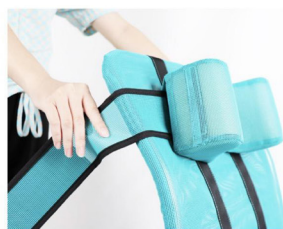

②




③

Chest belt



①

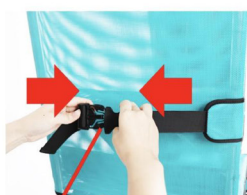

Buckle

②



③



④

Fixed pelvic Belt (M/L size)




①



②

Leg straps

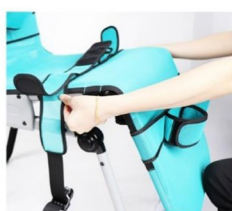

①



Buckle

②



③



④

Calf straps



Cleaning

- The bath chair frame can be wiped clean with a moist cloth and a mild cleaning agent.
- The bath chair cover is detachable and machine washable :
 - Wash with warm water (40°C/100°F) and use a mild detergent in a washing machine.
 - Place the bath chair cover inside a laundry bag to prevent damage to it and washing machine/dryer.
 - Tumble dry with low heat or dry. Do not bleach or iron. Do not dry clean.
- The bath chair accessories are washed by hand :
 - Wash with warm water (30°C / 85°F) and use a mild detergent, then leave to dry before putting back on the bath chair.
 - Do not tumble dry. Do not dry clean. Do not bleach or iron.

Maintenance

After using the bath chair, it is important to rinse with water to remove any remaining residue from soap or other bathing products.

Leave the bath chair to dry unfolded, and fold it after it is fully dried up for storage.

If the air is humid, store your bath chair and its accessories in a comparatively dry place and leave them to dry unfolded to prevent mold.

Continued and extended exposure to sunlight can change the colour of materials.

Table of symbols

	CE mark		Caution		Do not bleach
	Medical Device		Manufacturer		Do not iron
	Read the instructions before use		For use indoor		Machine wash-water temperature 40°C
	Single patient - multiple use		Authorized representative in the European community		Non-Sterile

BATH POSITIONING CHAIR SPECIFICATIONS	SIZES		
	SMALL	MEDIUM	LARGE
Seat depth	28 cm	35 cm	40 cm
Seat width	43 cm	43 cm	43 cm
Back height	49 cm	60 cm	71,5 cm
Calf length	25 cm	29,5 cm	34 cm
Overall length flat	100 cm	123 cm	142,5 cm
Overall width	43,8 cm	43,8 cm	43,8 cm
Seat height above floor	15-29,7 cm	15-33,8 cm	15-38 cm
Weight capacity	72 kg (160 lbs)	72 kg (160 lbs)	72 kg (160 lbs)

Warranty Statement

The vendor warranty period is 2 years, from the date of purchase as indicated on the sales receipt. This warranty does not cover product failure due to owner misuse or negligence. It also does not cover consumable items such as rubber parts, cover and other optional accessories, which are subject to normal wear and tear and need periodic replacement.

THE VENDOR WARRANTY WILL VOID IN THE FOLLOWING CIRCUMSTANCES:

- Customer made or arranged for modifications or repairs to be made without first having received written permission from the vendor.
- Defects resulting from neglect or an accident and/or from any use or maintenance other than that indicated in the user guide.
- Incorrect mounting or installation of third-party parts that are incompatible with the product.
- Improper storage / product care.
- Damage due to repairs carried out by unauthorized persons or due to improper disassembly of the product.
- Customer dissatisfaction with the product in terms of fit, weight, functionality, colour or design.

Warning

- Babies and young children are at risk of drowning when being bathed. Use extra caution when operating the bath chair in a bathtub.
 - Never leave the user unattended, even for a few moments. The carer shall at all times be in contact with the user.
 - Never let the user stand up in the bath chair.
 - Do not park or leave the bath chair on rough and uneven ground, or on an incline.
 - To prevent the risk of a bath chair catching fire, do not place it near an exposed heat source such as cigarettes, electricity, an open fire.
 - To avoid injury, ensure that the user is kept away when unfolding and folding this product.
 - Chest belt can cause strangulation, all the accessories must be carefully adjusted for the comfort and security of the user.
 - To avoid serious injury from falling or sliding on the bath chair, accessories should always be used.
 - Overloading, incorrect folding or the use of non-approved accessories may damage or break this bath chair.
- Maximum weight limits for all three sizes of bath chair : 160 lbs.
- The bath chair should be inspected regularly for signs of damage and wear. Do not use the bath chair if any part is broken, torn or missing. The damaged or worn parts must be repaired immediately.



1242 de Lanaudière
Joliette, Quebec
Canada J6E 3P1



1-866-739-2400



www.synetikergocare.com

Synetik

ErgoCare

R00

*Printed in Canada - 2025
Specifications and illustrations are subject to change without notice.*